HIP FIX™

LATERAL POSITIONING

EN	INSTRUCTIONS FOR USE	HIP FIX LATERAL POSITIONING
CS	NÁVOD K POUŽITÍ	LATERÁLNÍ POLOHOVÁNÍ
DA	BRUGSANVISNING	LATERAL POSITIONERING
DE	GEBRAUCHSANWEISUNG	LATERALE POSITIONIERUNG
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	ΠΛΑΓΙΑ ΤΟΠΟΘΕΤΗΣΗ
ES	INSTRUCCIONES DE USO	POSICIONAMIENTO LATERAL
FI	KÄYTTÖOHJEET	LATERAALINEN ASETTELU
FR	MODE D'EMPLOI	POSITIONNEMENT LATÉRAL
HU	HASZNÁLATI UTASÍTÁS	OLDALFEKVÉSŰ POZICIONÁLÁS
IT	ISTRUZIONI PER L'USO	POSIZIONAMENTO LATERALE
NO	BRUKSANVISNING	LATERAL POSISJONERING
PL	INSTRUKCJA UŻYCIA	POZYCJONOWANIE BOCZNE
PT	INSTRUÇÕES DE USO	POSICIONAMENTO LATERAL
SV	BRUKSANVISNING	LATERAL POSITIONERING

Explanation of symbols on labels and packaging

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Description

Hip Fix uses a unique 3-point fixation (crista iliaca and sacrum) to support the patient positioning, matching the surgeons need for space and manoeuvrability in the surgical field.

The Hip Fix models are equipped for use with single use or re-usable (one set per patient / procedure) cushions.

The devices are used to:


Facilitate accurate and reproducible patient and implant component orientation during procedures such as total hip replacement (THA).

Maintain stable lateral positioning of the patient throughout the procedure.

Ensure that the abdomen remains free from external pressure during the procedure.

The principles of 3 point fixation

Hip Fix uses a unique 3-point fixation (crista iliaca and sacrum) to support the patient positioning, matching the surgeons need for space and manoeuvrability in the surgical field.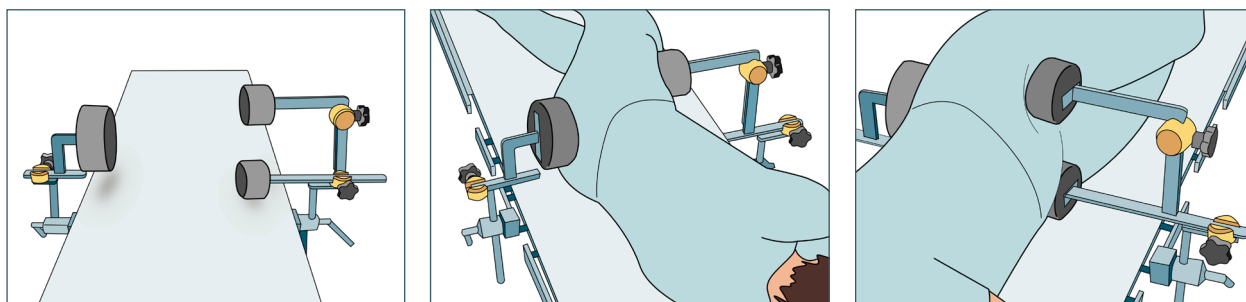

Intended use

Hip Fix Lateral Positioning Devices are non-sterile, non-active mechanical supports intended to provide accurate, stable and reproducible lateral positioning of a patient on the operating table before and during surgery.

Indication(s)

The Hip Fix will be specifically useable for interventions on the hip joint (pelvis) where the patient is positioned laterally and where a strong fixation of the patient is desired.

General surgical procedures involving lateral positioning.

Lateral patient positioning where the positioned angle of the patient relative to the OR table surface is desired to be variable.

Lateral patient positioning where the anatomy of the patient is a challenge for the OR staff.

Contraindication(s)

Do not use on damaged skin.

Do not use in patients where the iliac crest cannot be felt/located.

Do not use in Surgical procedures where the perioperative manipulation of the leg involves / demands an angle of more than 90 degrees relative to the median of the patient.

Intended users

Trained operating room (OR) staff, including orthopaedic surgeons and scrub nurses, and other theatre personnel who have received training in assembling, mounting, adjusting and using the Hip Fix device. The device is not intended for lay users or use outside professional healthcare facilities.

Intended patient population

Patients undergoing surgery requiring lateral positioning, including but not limited to total hip arthroplasty and other hip procedures.

Adult patients.

Warning

- Follow section cleaning, disinfection and storage instructions to avoid damage of device, malfunction and/or cross contamination.
- Avoid conditions of electro discharge.
- By multiple use of single use cushions, a risk of cross contamination cannot be excluded.
- Do not lean on or against the support, leaning on the support may damage the device and may result in sliding of cushions, patient injury and/or skin damage.
- Check all parts before usage and replace parts that show signs of wear and tear to ensure full functionality.
- Check for any wear of the operating table attachments so that they do not negatively affect the use and mounting of Hip Fix.

Precaution


- Assembling, mounting and adjusting of the device should only be performed by qualified medical professionals.
- Only tighten handles and knobs by hand.
- When using rail clamps check whether they are usable for a 25 x 10 mm (1" x 1/4") table rail and will adapt the octagonal rods (4 flat sides) of the support. The manufacturer is not responsible for events related to the use of rail clamps other than recommended.
- Patient positioning and fixation should be done by at least two qualified medical professionals.

Adverse effects

Patients with impaired wound healing ability and/or impaired blood circulation can develop pressure marks and pressure marks easier.

Hip Fix models/configurations

	Part name	Part number	60801200	60801100	60801202	60801102	60801201	60801101
			Hip Fix 2000, Complete set with reusable cushions	Hip Fix 1900, Complete set with reusable cushions, adjustable arm	Hip Fix 2015, Complete set with single-use cushions	Hip Fix 1915, Complete set with single- use cushions, adjustable arm	Upgrade Set from reusable to single use cushions	Upgrade set from reusable to single use cushions, adjustable arm
1	Back support	200010	x					
2	Front support, lower	200014	x					
3	Front support, upper	200017	x					
4	Support cushion (large) reusable	2000-02	x	x				
5	Support cushion (small) reusable	2000-01	x	x				
6	Front post	200021	x		x			
7	Rear post	200026	x		x			
8	Front post	190008		x		x		
9	Clamp horizontal	190004		x		x		
10	Rear post	190009		x		x		
11	Rear support	190012		x				
12	Front support lower	190016		x				
13	Front support upper	190015		x				
14	Font support	191513				x		x
15	Back support	191510				x		x
16	Back support	201510			x		x	
17	Font support, lower	201514			x		x	
18	Front support, upper	201517			x		x	
19	Cushion Holder front upper	2015-10			x	x	x	x
20	Cushion Holder front lower	2015-20			x	x	x	x
21	Cushion Holder back	2015-30			x	x	x	x

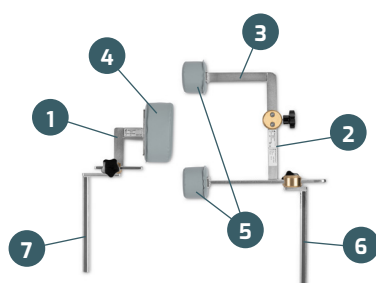Hip Fix 2000 # 60801200

Classic Hip Fixation with a fixed distance to the iliac crest from the front post cushions. Secures 90 degree angling of the patient. Fixed foam cushions, screw mounted.


Hip Fix 2015 # 60801201

For existing Hip Fix 2000 users. This set upgrades – in a very simple way – the "classic" Hip Fix 2000 to the new version, ready for single use cushions.

Hip Fix 2015 # 60801202

Contains the Upgrade Set 2015-100, and 2 posts for the OR table.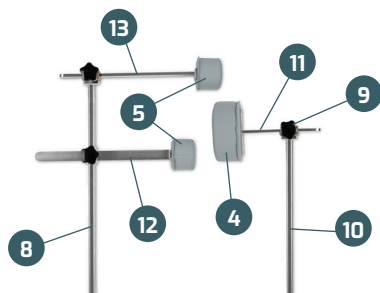

Hip Fix 1900 # 60801100

Individual vertical adjustable front cushions. Patient angle to the table variable. Supports bariatric patients. Fixed foam cushions, screw mounted.

Hip Fix 1915 # 60801101

For existing Hip Fix 1900 users. This set upgrades – in a very simple way – the "classic" Hip Fix 1900 to the new version, ready for single use cushions.

Hip Fix 1915 # 60801102

Contains the Upgrade Set 1915-100, and 2 posts for the OR table.

Preparation and inspection:


1. Inspect if the clamp fixation screws are working and not damaged.
2. Inspect if the rail clamps fit properly on the table rail.
3. Inspect if the rail clamp fits properly on the vertical stems of the Hip Fix.
4. Replace the rail clamp(s) if not working properly.
5. Inspect the Hip Fix components for damage.
6. Inspect the Hip Fix for proper function of the joints and clamps.
7. If damage is noticed do not use the device.
8. Clean items.

If you are using a Hip Fix model with single-use cushions


HOLDER & CUSHION ASSEMBLY


Front lower support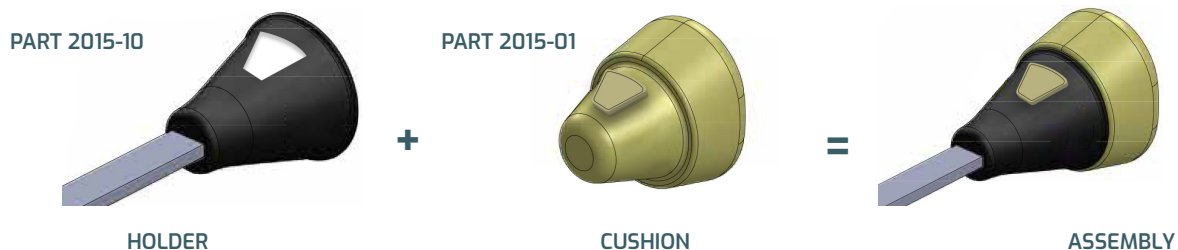

Front upper support

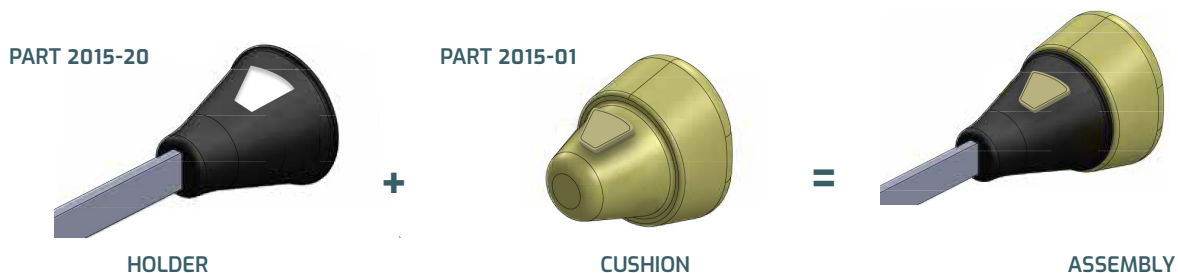
Back support

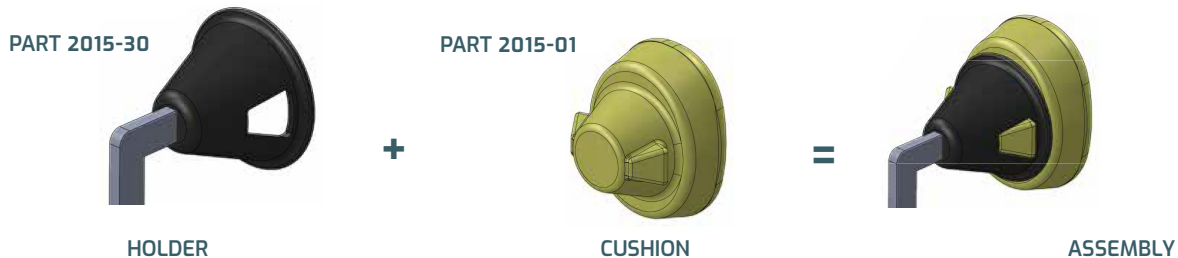HIP FIX ASSEMBLY


All models are to be used with rail clamp with plane studd

1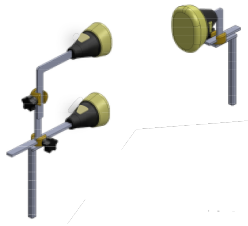


Begin by assembling the front support and back support * SEE NEXT PAGE

2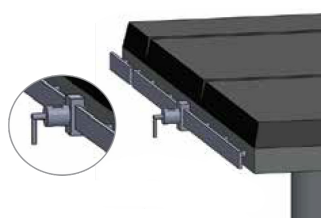


Apply the 2 clamps on each side, centrally.

3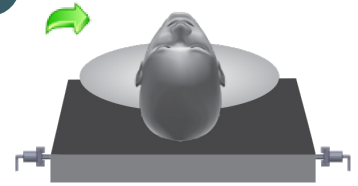


Place the patient in horizontal position on the operating table.

4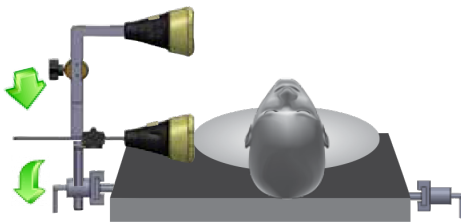


Place the front support assembly in the table clamp.

5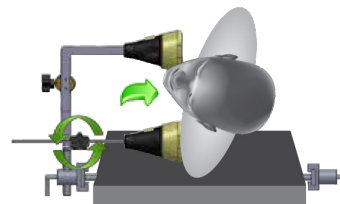

Tilt the patient a little backwards. Adjust the lower front support by loosening the knob and pushing the support inwards so that the cushion is on the lower crista iliaca. Tighten the handwheel.


6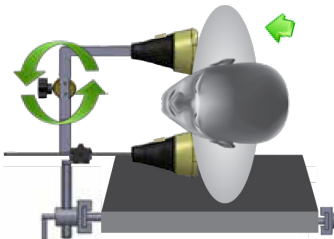

Tilt the patient back up towards the upper front cushion.


Adjust (on the upper knob) the upper front support so it supports the upper crista iliaca.

7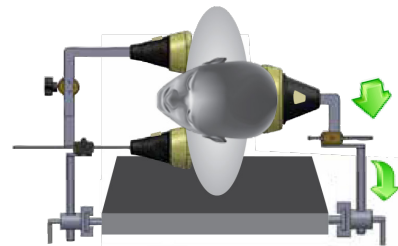

Mount the rear support assembly on the Hip Fix clamp.


8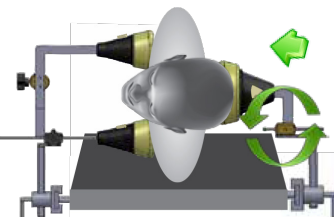

Slide / adjust the back support to fully support the patient on the sacrum.

Then tighten the knob.

9

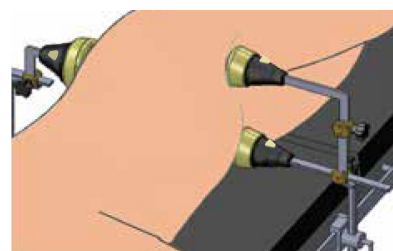Test stability of the positioning and drape according to local procedures.

10


Avoid friction.

11

Do not apply to damaged skin.

12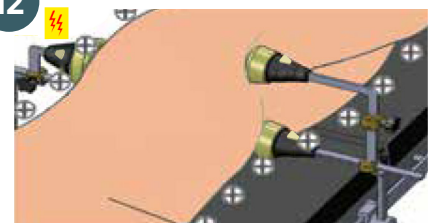

Avoid conditions of electro discharge.

Instruction videos can be found here: mediplast.com/discover/hip-fix

mediplast.com

*** Clinical Option: HIP FIX 60801200 & 60801202**

First assemble the front support, lower by sliding it into the front post brass coupling. Tighten the lower knob by hand. Slide the front support, upper into the double brass coupling on the front support, lower and lock the connection with the upper knob by hand.

*** Clinical Option: HIP FIX 60801100 & 60801102**


First assemble the front support by sliding Front support lower and Front support upper into the front post. Tighten the knob by hand. Assemble Back support by sliding the rear/back support into the rear post. Tighten knob by hand.

Removing the Hip Fix

1. Remove the sterile sheets / draping.
2. Hold the patient in a secured position laterally. Removing the lateral support has to be done by two persons. One has to support the patient, and the other one is releasing the locking knobs and rail clamps
3. Release the complete back support (including the rail clamp) and gently remove it with the patient still in the lateral position.
4. Support / tilt the patient into a safe and appropriate position on the back (supine).
5. Remove the single use back pad from the holder if fitted and discharge.
6. Remove the two single use cushions on the front support if fitted and discharge.
7. Release the complete front support (including the rail clamp) by the rail clamp and gently remove.

De-assembling Hip Fix

1. Remove the single use cushions (if fitted) by depressing the tabs in the cut-outs of the holders. Then pull the cushion out of the holder. The cushions are single use products.
2. Back support: Release the rail clamp from the support. By means of a quarter turn on the black handwheel, open the brass coupling and slide the support out of the coupling.
3. Front support: Release the rail clamp from the support. By means of a quarter turn on the two black handwheels, open the brass couplings and slide the two support parts out of the couplings.

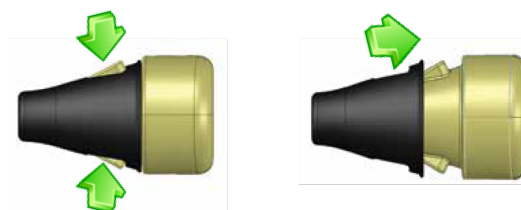Cushion disassembly for cleaning and inspection of holders.

CLEANING, DISINFECTION AND STORAGE OF THE DEVICE

Instructions for Care, Cleaning and Disinfection Hip Fix Lateral positioners (applies to all models).
The frequency of routine cleaning will depend on the use of the device.

Cleaning / Disinfection:

Please observe the following before starting cleaning / disinfection procedures:

Use suitable personal protective measures to protect yourself from stains. Place the device on a suitable and stable surface.

Cleaning

- Use a clean soft cloth with warm (max. 65° C) water and a mild, neutral soap solution and wipe / wash the parts gently – avoid excessive fluid in the areas around the screws, threads and fixed cushions (if mounted).
- Wipe the parts thoroughly with a soft cloth and clean water.
- Wipe the parts dry thoroughly with a soft cloth or paper napkins.
- Assemble the device (do not mount any single use cushions until just before a surgical procedure).

Disinfection

If disinfection is needed repeat steps a) to d) while using a solution of Sodium Hypochlorite (dilution 1/100 – 10.000 ppm) or a standard disinfectant product (suited for OR equipment) including products containing benzalkonium chloride.

On the plastic parts (black single use pad holders), a 70% isopropyl alcohol solution should be used to avoid discoloration and material fatigue due to Sodium Hypochlorite.

Storage

Store the parts in an appropriate place preferably transport box in which the product was delivered.

Copies of this instruction for use can be ordered free of charge. Contact us:

Disposal of device according to local routines in medical facilities.

In case of any serious incident that has occurred in relation to the device, please report to the manufacturer and authority of the member state in which the user/or patient is established.

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Medioplast AB